BRUKSANVISNING HURTIGTEST

Utstyr

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Rapid test instruction:

Equipment:

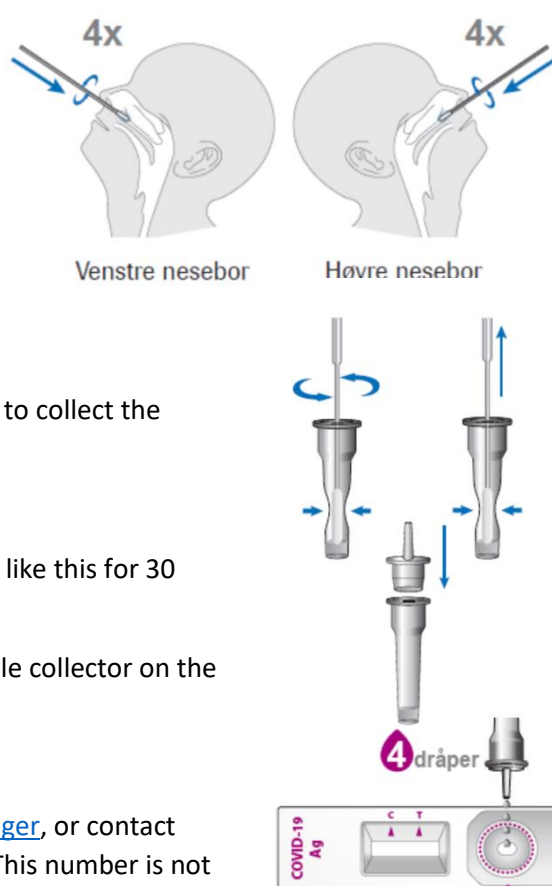
- Nasal swab
- Test device
- Tube containing buffer liquid
- Clear cap for tube

Video: <https://youtu.be/LDusiOf3Zo0>

Start by unpacking the equipment included in the bag. Lay it on a clean surface. Remove the test device from its packaging, then the nasal swab and perform the test.

- 1) Slightly tilt the patients head back.
- 2) Roll the nasal swab stick whilst you are entering the nostril, around 2 cm in from the edge of the nostril.
- 3) Rotate the nasal swab 4 times against the inner walls of the nose. Repeat this for both nostrils.
- 4) Remove the film from the test tube containing buffer fluid and place the nasal swab into the tube. Pinch the bottom of the tube and roll the swab at least 10 times.
- 5) Remove the nasal swab whilst pinching the sides of the tube to collect the absorbed liquid from the nasal swab.
- 6) Firmly attach the nozzle cap onto the tube.
- 7) Carefully flip the test tube, so the cap is facing down. Keep it like this for 30 seconds.
- 8) Slowly drip 4 drops of the liquid from the tube into the sample collector on the test device.
- 9) Read the results after 15-30 minutes.
- 10) Register positiv test at <https://hjemmetest.remin.no/stavanger>, or contact Stavanger Kommune Covid-19 tracking by calling + 47781641. This number is not an information line and is only to be used in reporting positive test results.

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