# BRUKSANVISNING HURTIGTEST

## Utstyr

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## Rapid test instruction:

### Equipment:

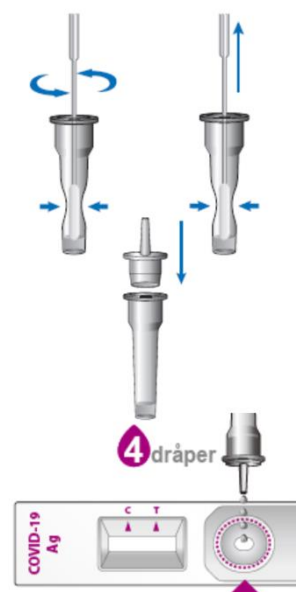
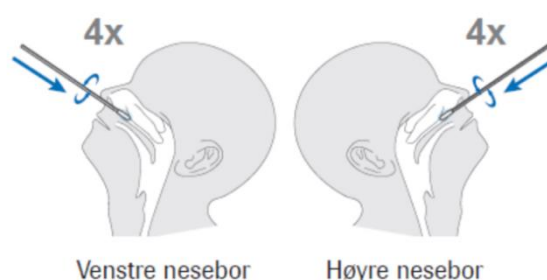
- Nasal swab
- Test device
- Tube containing buffer liquid
- Clear cap for tube

Video: <https://youtu.be/LDuslOf3Zo0>

Start by unpacking the equipment included in the bag. Lay it on a clean surface. Remove the test device from its packaging, then the nasal swab and perform the test.

- 1) Slightly tilt the patients head back.
- 2) Roll the nasal swab stick whilst you are entering the nostril, around 2 cm in from the edge of the nostril.
- 3) Rotate the nasal swab 4 times against the inner walls of the nose. Repeat this for both nostrils.
- 4) Remove the film from the test tube containing buffer fluid and place the nasal swab into the tube. Pinch the bottom of the tube and roll the swab at least 5 times.
- 5) Remove the nasal swab whilst pinching the sides of the tube to collect the absorbed liquid from the nasal swab.
- 6) Firmly attach the nozzle cap onto the tube.
- 7) Carefully flip the test tube, so the cap is facing down. Keep it like this for 30 seconds.
- 8) Slowly drip 4 drops of the liquid from the tube into the sample collector on the test device.
- 9) Read the results after 15-30 minutes.
- 10) If the test is positiv: Contact the Stavanger Kommune Covid-19 tracking by calling + 47781641. This number is not an information line and is only to be used in reporting positive test results.

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